

N95 & KN95 MASK - INSTRUCTIONS FOR USE AND STORAGE

- Use one (1) mask per employee
- Use one mask for a MAXIMUM of three (3) shifts
- Thoroughly wash hands:
 - Before putting on the mask
 - Before adjusting the mask on your face
 - Before taking off the mask
- Only touch the straps when putting the mask on or taking the mask off
- After each use, wipe the mask with a disinfecting wipe
- The mask should be removed and discarded if:
 - Soiled
 - Damaged, or
 - Hard to breathe through after multiple uses
- Store cleaned mask in a paper bag, labeled with your name
- **DO NOT** share mask
- After three (3) shifts, if mask is undamaged and breathable, return to supervisor for cleaning



REMEMBER to **thoroughly wash your hands** throughout the day. It is one of the best ways to decrease the spread of COVID-19!